---

An AK MEDICAL Company

---

**JRI ORTHOPAEDICS LTD**

18 Churchill Way,  
35A Business Park,  
Chapelton,  
Sheffield,  
S35 2PY, England

Instructions for Use

# Instrument Re-processing

---

English .....	3
Česky.....	7
Dansk .....	11
Deutsch.....	15
Español .....	20
Français.....	25
Italiano.....	30
Nederlands .....	35
Norsk .....	40
Polski .....	44
Português .....	49
Suomi.....	54
Svenska.....	59

**RETURN TO:**

**FAO: CUSTOMER SERVICE DEPARTMENT**

**JRI ORTHOPAEDICS LIMITED**

18 Churchill Way  
35A Business Park  
Chapelton  
Sheffield S35 2PY

An AK MEDICAL Company

**Website:** [www.jri-ltd.co.uk](http://www.jri-ltd.co.uk)

**Tel:** 0114 345 0000

**Fax:** 0114 345 0004

I confirm that all instrumentation in this container has been cleaned and sterilised in accordance with the instructions on the following pages.

<b>Hospital:</b>	<b>Method of Sterilisation:</b>
<b>Method of Cleaning</b> (Delete as appropriate):  Manual / Automated Wash / Both	
	<b>Machine Cycle No:</b>

<b>Name:</b>	<b>Position:</b>
<b>Signature:</b>	<b>Date:</b>

**Note**

It is the responsibility of the reprocessor to ensure that reprocessing is performed using the appropriate equipment and materials, and that the personnel in the reprocessing facility have been adequately trained to achieve the desired result. This normally requires validation and routine monitoring of the process.

Likewise any deviation by the reprocessor from these instructions should be properly evaluated for effectiveness to avoid potential adverse consequences.

# Cleaning, Decontamination and Steam Sterilisation of Orthopaedic Surgical Instruments (In accordance with ISO 17664-1:2021)

## INSTRUCTIONS

### **Do not exceed 138°C for products that contain plastic components.**

Repeated processing, as described below, has minimal effect on JRI re-usable manual instruments. End of life is normally determined by wear and damage due to use.

### ***Pre-operative cleaning and decontamination***

**NOTE: For instruments that have been previously cleaned, and for which a valid decontamination certificate is available, manual cleaning, prior to use, is not necessary, provided that a validated automated cleaning system is used.**

***Automated Cleaning*** A validated automated cleaning system should be used in accordance with the appropriate Part of HTM 01-01, to prepare instruments for clinical use. If applicable, instruments should be loaded so that hinges are open, and cannulations and holes can drain effectively.

#### ***Automatic Washing Cycle Steps and Parameters***

Load the devices into the automatic washer-disinfector.

- Connect any device lumen(s) to the rinsing ports of the injector basket.
- Place blind holes at a downwards incline for drainage.
- Place all instruments in the open position whenever possible (i.e. hinged/mating features, articulating surfaces).

Minimum cycle parameters:-

- Pre-cleaning: 20°C for 5 minutes (Deionised Water)
- Main Wash 1: 40°C for 10 minutes using a PH neutral detergent
- Main Wash 2: 20°C for 2 minutes using a PH neutral detergent
- Neutralising/ Rinse 2: 20°C for 1 minute (Deionised Water)

### ***Post-operative cleaning and decontamination***

**Do not allow instruments to dry post-operatively for a period of greater than 15 minutes as this has been shown to increase the risk of residual protein contamination.**

Excess soiling should be removed with disposable cloth/paper wipes and all instruments covered with a damp cloth. It is recommended that instruments be reprocessed as soon as is reasonably practical, and that they be transferred directly to the cleaning facility following use. Cleaning agents with chlorine or chloride as the active ingredient are corrosive to stainless steel and must not be used.

Long narrow cannulations, blind holes and screw-threads require particular attention during cleaning.

### ***Preparation for cleaning***

It is recommended that a neutral pH cleaning solution is used at a temperature and dilution recommended by the solution manufacturer. Highly alkaline solutions (>pH 9.0) should be avoided, as these may damage metal components and affect their function. Acidic solutions (<pH 6.0) **should not** be used, as they will cause corrosion and device malfunction.

If specialist cleaning tools are required, these should be made available.

**Instruments that can, MUST be disassembled prior to cleaning, in accordance with the instructions provided with the instruments.**

### ***Pre-Cleaning***

**Note: For severely soiled instrument, rinse under a cold tap for a minimum of 1 minute to remove any gross contaminants.**

### ***Manual Cleaning***

**Manual cleaning is recommended where there is visible soiling still evident on the instruments, or where pre-cleaning may not have been effective, such as cannulated, or threaded, instruments, or instruments with blind holes or complex component parts, where disassembly is not possible.**

**Equipment:** Use a neutral pH enzyme, low foaming, non-ionising solution and a soft bristled brush. Water for injection, or purer, to be used throughout.

**Method:** Pre-soak in pH neutral, low foaming, non-ionising solution for a minimum of 20 minutes at room temperature. Whilst the devices remain submerged, scrub all surfaces using a soft nylon brush until all visible blood debris has been removed. Ensure that hinged instruments are cleaned in both open and closed positions.

Ensure that all instrument surfaces are visibly clean and free from stains and tissue.

Inspect all instrument to ensure proper function. Forceps tips should be properly aligned. Cutting instruments and knife blades should be sharp and undamaged.

**Note: Cannulations and holes to be cleaned using an appropriate brush, ensuring that full depth of the instrument is reached.**

Once free from visible soiling, place in a fresh pH neutral enzyme, low foaming, non-ionising solution and sonicate for 10 minutes.

Following sonication, rinse instruments for at least 1 minute using purified water. Ensure that running water passes through cannulations, and that blind holes are repeatedly filled and emptied.

Instruments should be dried with a clean, disposable, absorbent, non-shedding wipe and allowed to dry in ambient air for a minimum of 10 minutes before packing.

**Note: Personal protective equipment, in accordance with local health and safety procedures, should be worn during the cleaning procedure.**

### ***Automated Cleaning***

A validated automated cleaning system should be used, in accordance with the appropriate Part of HTM 01-01, to prepare instruments for clinical use. If applicable, instruments should be loaded so that hinges are open and cannulations and holes can drain effectively.

The run-cycle should include at least rinse, wash, rinse and disinfection stages. A typical run cycle should include:-

- Pre-cleaning: 20°C for 5 minutes (Deionised Water)
- Main Wash 1: 40°C for 10 minutes using a PH neutral detergent
- Main Wash 2: 20°C for 2 minutes using a PH neutral detergent
- Neutralising/ Rinse 2: 20°C for 1 minute (Deionised Water)

Cannulations, holes etc. should be checked when unloading, to ensure complete removal of visible soil. If necessary, repeat cycle or use manual cleaning. All parts should be thoroughly dried after cleaning.

### ***Automatic Washing Cycle Steps and Parameters***

Load the devices into the automatic washer-disinfector.

- Connect any device lumen(s) to the rinsing ports of the injector basket.
- Place blind holes at a downwards incline for drainage.
- Place all instruments in the open position whenever possible (i.e. hinged/mating features, articulating surfaces).

### ***Disinfection***

Disinfection should only be used as a part of a full steam sterilisation process and **should NOT** be the sole method of preparing surgical instruments for clinical applications.

Disinfection should be carried out in accordance with manufacturer's instructions (for automated systems) and appropriate HTM 01-01 guidelines as described below.

### ***Inspection / Functional Testing***

Cleaned and/or disinfected instruments should be inspected to

- Ensure that all visible blood and soiling has been removed
- Ensure that the devices are not damaged, worn or deformed/twisted
- Ensure the smooth operation of any moving parts over the required range of motion

- Ensure that component parts of larger assemblies can be readily assembled and that function is not compromised.
- Ensure that there are no chemical residues remaining from the cleaning / disinfection process

### **Maintenance**

A small quantity of surgical grade instrument lubricant can be used to reduce friction and wear on hinges, threads and moving parts.

### **Packing**

Individually – A standard polypropylene/ Tyvek (or equivalent) sterilisation pouch of the appropriate size is to be used to ensure that the instrument does not stress or tear the packaging seals. Sets – Instrument sets should be loaded into the dedicated instrument trays, or general-purpose instrument trays for sterilisation. If applicable, standard medical grade steam sterilisation wrap should be used, following the recommended AAMI double-wrap method (ANSI/AAMI ST-79:2017)

### **Sterilisation**

#### **Instruments can be sterilised in the assembled condition**

Steam Sterilisation should be conducted in a validated vacuum autoclave (BS EN 285:2015+A1:2021) operating at 134 -137 °C for a minimum holding time of 3 minutes, in accordance with the autoclave manufacturer's recommendations.

Steam sterilisation of multiple instruments in a single cycle should not exceed the loads recommended by the autoclave manufacturer.

**Where TSE/vCJD contamination is suspected, a steam sterilisation cycle of 18 minutes at 134°C is recommended by the World Health Organisation (WHO/CDS/CSR/APH/2000.3)**

### **Additional Information**

Sterile, packaged instruments should be stored in a designated, limited access, area that is well ventilated and provides protection, as far as possible, from dust, moisture, insects, vermin, and extremes of temperature and humidity.

Sterilised instrument packages should be checked, prior to opening, to ensure that the packaging has not been compromised.

### **References**

- ANSI/AAMI ST79:2017 - Comprehensive Guide To Steam Sterilization And Sterility Assurance In Health Care Facilities
- BS EN 285:2015+A1:2021 – Sterilisation - Steam Sterilisers - Large Sterilisers
- Health Technical Management Memorandum 01-01: Management and Decontamination of Surgical Instruments (medical Devices) Used in Acute Care
- ISO 17664-1:2021 – Processing of health care products — Information to be provided by the medical device manufacturer for the processing of medical devices — Part 1: Critical and semi-critical medical devices
- ISO 15883-5:2021 – Washer-disinfectors — Part 5: Performance requirements and test method criteria for demonstrating cleaning efficacy
- WHO Infection Control Guidelines for TSE (WHO/CDS/CSR/APH/2000.3, March 1999)
- ACDP-TSE Annex C – General Principles of Decontamination and Waste Disposal

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